

Ahygrol®

Liquid for degreasing and dehydrating

Prescription

Ahygrol used for degreasing and drying cavities and root canal before filling or tooth stump before installing fixed prosthesis and for cleaning surfaces of the prosthesis from contamination before fixation.

Composition and main characteristic

Ahygrol is a mixture of volatile organic solvents, indifferent to the surrounding soft tissues of the oral cavity, without causing a painful reaction in contact with mucous membranes.

Easy soluble in water. Drying time 10 sec. **Ahygrol** also contains a natural surface active agent which improves wettability desorption of water and traces of grease.

Ahygrol liquid application allows you to quickly and accurately degrease and dry the surface of the tooth and the prosthesis before fixation.

Method of application:

Degreasing and drying performed after dissection, washing with water and drying of cavity or stump of the tooth, the inner surface of the non-removable denture.

A cotton pad soaked with liquid

Ahygrol or use the pipette handle cavity or surface of the tooth and the denture.

After that on the treated surface directed weak jet of compressed air through which the fluid layer is displaced to the periphery of the surface, by this removing maximum possible water and traces of grease.


Attention! **Ahygrol** is a flammable liquid. When working with it don't use an open flame or electric heaters with an open spiral.


Storage conditions and labeling

Storage conditions

Ahygrol have to be stored in packing inside the warehouse and to be protected from precipitation and direct sun light, to be far minimum 1 m. from heaters.

Labeling

The  symbol on the package show expire date.

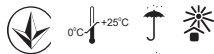

Latus PE, 61010 Textylina str.82, Khrkov, Ukraine.
Tel +380577332697. www.latus.com.ua