

# Endocort®

## Zinc oxide eugenol cement

### Prescription

**Endocort** used for permanent tooth fillings canals of all group of teeth or a single material or by using a pin.

### Composition and main characteristic

**Endocort** is a filling material class "powder-liquid." The powder composition comprises zinc oxide, barium sulfate, hydrocortisone acetate, dexamethasone, paraformaldehyde and additives di iodum thymol preventing entrance of formaldehyde from the root canal and increased coloring material. The liquid used for mixing-eugenolum. Working time of the material is 2-6 hours. Curing of material in the tooth canal about 24-48 hours. If necessary, the cured material can be easily removed from the canal.

### Method of application:

The root canal is treated in a conventional manner by instrumental and medical. Dried by compressed air.

To a mixing pad or piece of glass put powder and liquid in ratio 2:1 by volume. The components are


mixed thoroughly for 20-30 seconds until get a smooth paste. With the help of endodontic files and reamers fill the resulting paste root canal. After that you need to pass the X-ray of control of filling and then begin to tooth fillings. Attention! After selecting the powder portion to close the bottle with cap.


### Storage conditions and labeling

#### Storage conditions

**Endocort** in packaged form should be stored in closed, cool place, protected from rain and direct sunlight at a distance of at least 1 meter from heaters.

#### Labeling

The  symbol on the package show expire date.

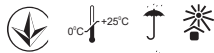

Latus PE, 61010 Textilna str.82, Khrkov, Ukraine.  
Tel +380577332697. [www.latus.com.ua](http://www.latus.com.ua)