

Hemalat[®] forte

Hemostatic agent

Prescription

Hemalat forte is used in therapeutic dentistry like a hemostatic agent in the case of capillary bleeding from the gums, root canal with vital pulp extirpation.

Composition and main characteristic

Hemalat forte prepared on the base of iron sulfate. **Hemalat forte** is well suited for hemostasis and control of gingival liquid in the case of fixing indirect adhesive restorations and treatment. In a short time it stops bleeding, removes the pollution caused by the circulation of gingival liquid, given by this an optimal bonding.

Very gentle action on the hard and soft tissue. Well washed off by water.

Method of application

In the case of capillary bleeding from the gums on a sterile cotton swab is applied **Hemalat forte** and smeared bleeding surface to a complete stop bleeding.

In the case of bleeding from the root canal put **Hemalat forte** on a cotton turunda directly into the root canal.

By this also removes excessive

amounts of coagulated blood, which create clots out of capillary holes.

To coagulate blood does not adhere to the treated tissue during application **Hemalat forte** additionally spray water using a saliva ejector.


Then **Hemalat forte** washed by water, dried gum by air stream and carried out the necessary manipulations.


Storage conditions and labeling

Storage conditions

Hemalat forte have to be stored in packing inside the warehouse and to be protected from precipitation and direct sun light, to be far minimum 1 m. from heaters.

Labeling

The  symbol on the package show expire date.

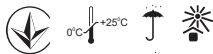

Latus PE, 61010 Textylna str.82, Khrkov, Ukraine.
Tel +380577332697. www.latus.com.ua