

Resodont[®]

Cement resorcinol-formalin

Prescription

Resodont used for filling the root canal after extirpation of the pulp and for treatment of periodontitis.

Composition and main characteristic

Resodont is a resorcinol-formalin chemical curing type material powder-liquid. The kit includes a powder, liquid №1 and liquid №2. By mixing all the components formed the paste is, retaining the plasticity within 2 hours and hardening within 24 hours.

The cured material has a disinfectant and antiseptic properties, radiopaque.

Note: Not recommended to use the cement for filling the canals of front tooth, because it is possible coloration of crown part of the tooth.

Method of application:

The root canal is treated in a conventional manner of instrumental or medication. Carefully dried by cotton turundus or paper points.

Onto a mixing pad or glass and dig liquid №1 and liquid №2 in ratio of 1:1, mixed added powder to get


plasticity paste. With the help of endodontic files and reamers or root needle filled with the resulting paste root canal. Make X-ray control of the filling channel.


Storage conditions and labeling

Storage conditions

Resodont in packaged form should be stored in closed, cool place, protected from rain and direct sunlight at a distance of at least 1 meter from heaters.

Labeling

The  symbol on the package show expire date.

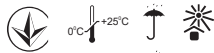

Latus PE, 61010 Textilna str.82, Khrkov, Ukraine.
Tel +380577332697. www.latus.com.ua