

Tempolat[®]-C

Self curing cement for temporary crowns

Prescription

Tempolat-C is designed for use in prosthetic dentistry for the fabrication of temporary crowns and bridges, inlays, onlays and coatings.

Composition and characteristics

Tempolat-C is a composite material chemical curing type paste-paste. The composition of the material consists of a multifunctional methacrylate oligomers and finely divided filler. Optimal fluidity Tempolat-C material provides convenience in making an impression. When mixing the main and the catalyst paste in a ratio of 1: 1 (by volume) mixture obtained stores working time 3-4 minutes (21°C temperature).

Method of application

Before preparing a fragment or a bridge made impression from silicone material Consiflex. Dry the prepared teeth and lightly smeared with Vaseline or a similar product to separate the fragments together with the surrounding tissue, and if there are composite fragments of superstructure. Extruded from the syringe onto a mixing pad equal in volume and amount of the main and catalyst pastes were mixed for 30 seconds. Fill the impression or the plastic matrix by material Tempolat-C, causing the material to the occlusal surface of first and then in the gingival region, slightly overlapping them. Install and fix with gentle pressure on the teeth restored, filled by material Tempolat-C impression or plastic matrix. Time from the start of mixing to placement in the mouth should not exceed 3 minutes. After 5-6 minutes from the start of mixing material, removed construction from the oral cavity, and the material must be in an elastic state. Not earlier than 7-10 minutes after removal construction from the oral cavity can make processing and polishing the crown or bridge.

Recommendations

To increase the strength of provisional

restoration take off interdental septum. If necessary (if insufficient space) should increase the desired areas, cutting the impression material. For easy removal from the oral cavity should be closed undercut. When a tooth is missing or there is a gap in the molar area you need to cut a furrow to create a stable intermediate of the provisory restoration. If necessary, before removing the impression close the place the of the missing tooth with a plastic from a standard kit. To fix an artificial tooth to the adjacent teeth with wax.

Instead of the impression can be used laboratory-made mouth guard, caps for single crowns, polycarbonate-blank crowns. If necessary, you need to make an appropriate grinding instrument.

To fix the construction you can use any temporary cement.

Check occlusion.


Conditions of storage and labeling

Condition of storage

Tempolat-C in packaged form should be stored in closed, cool place, protected from atmospheric precipitation and direct sunlight. To increase the shelf life prolonged storage it is recommended to use a refrigerator at + 5°C. Before the work necessary to sustain Tempolat-C for at least 2-3 hours at room temperature to recover the ductility of the paste.

Labeling

The ⏰ symbol on the package show expire date.

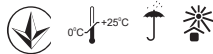

Latus PE, 61030, Textylna str.82, Khrkiv, Ukraine.

Tel +380577332697. www.latus.com.ua