

# Tempolat®-F

## Temporary fixing cement

### Prescription

Tempolat-F paste used for temporary fixation of crowns, bridges and other fixed prosthesis.

### Composition and characteristics

Paste Tempolat-F is made on-base zinc sulfate cement which contain oil-polymer paste-forming agent. The paste has no taste, well tolerated by the tooth and gums. Pasta does not contain eugenolumum.

Before use, it is not necessary to dry the stump and the crown, as the paste cured in the presence of moisture.

### Method of application

Tempolat-F put to the crown and fixed to a tooth stump for 7-10 minutes. Excess of the paste is removed before curing with a cotton swab. The cured material is sufficiently durable and can be used for a long time fixation, thus can be easily removed without damaging the hard tissues of the tooth stump.


### Conditions of storage and labeling

#### Condition of storage

Tempolat-F in packaged form should be stored in closed, cool place, protected from rain and direct sunlight, at a distance of not less than 1 m from heaters devices.

#### Labeling

The ⏰ symbol on the package show expire date.

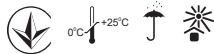

Latus PE, 61030, Textylna str.82, Kharkiv, Ukraine.  
Tel +380577332697. [www.latus.com.ua](http://www.latus.com.ua)