

Tempolat®-SC

**Temporary composite
zinc-sulphate cement**

Prescription

Tempolat-SC used in dentistry practice for temporary filling in endodontics treatment, coating of medical products in caries cavity, temporary fixing of crowns and bridges.

Composition and characteristics

Tempolat-SC is a white powder of temper cement on zinc sulphate base. When mixed with water powder Tempolat-SC create a movable paste that within a short time (2 - 3 minutes) come thick and hardens. Tempolat-SC have good adhesion to the tooth, and does not shrink substantially. Cement contains bactericidal additive chinosolum.

Method of application

On a glass plate (preferably pre-cooled) was placed a small portion of the powder (about 0.5 g) was added drop wise and the next 6 drops of water. Within 15-20 with powder 2-3 portions are added to the water and immediately mixed rubbing movement by modeling knife to form a paste. Gently blowing air to cavity, using a burnisher fill it with cement. Then


paste smooth moist with a cotton swab. The seal is cured about 3 minutes. More 3-4 minutes are required to set the primary seal hardness and water resistance. During hardening the seal can not be correct or be thicken, as this may lead to the destruction of the cement.


Conditions of storage and labeling

Condition of storage

Tempolat-SC in packaged form should be stored in closed, cool place, protected from rain and direct sunlight.

Labeling

The  symbol on the package show expire date.

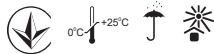

Latus PE, 61030, Textylna str.82, Kharkiv, Ukraine.
Tel +380577332697. www.latus.com.ua