

# Trimlat®

## Liquid for processing root canals

### Prescription

Trimlat liquid used for processing of root canal by softening dentin hydroxyapatite by chelator EDTA and subsequent mechanic processing by instruments to get the desired form of the root canal.

### Composition and main characteristic

Trimlat contains EDTA - (15%), thickening additive, a detergent and water.

Trimlat liquid like lubrication facilitates the movement of files and chemically acting expands the root canal. Liquid remains easily rinsed by water.

### Method of application:

Pour out drop by drop of Trimlat liquid(2-3 drops) on the pad for mixing or glass plate. Collect drops by cotton turunda and transferred Trimlat to the mouth of the root canal. Leave for 2-3 minutes and begin mechanical expansion.

Further canal thoroughly washed by water, treated by a solution of sodium hypochlorite or 3% hydrogen peroxide solution, then dried with compressed air.


After treatment you can begin to fill the canal by selected filling material.


### Storage conditions and labeling

#### Storage conditions

Trimlat have to be stored in packing inside the warehouse and to be protected from precipitation and direct sun light, to be far minimum 1 m. from heaters.

#### Labeling

The  symbol on the package show expire date.

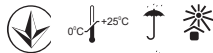

Latus PE, 61010 Textilna str.82, Khrkov, Ukraine.  
Tel +380577332697. [www.latus.com.ua](http://www.latus.com.ua)