

Desgel[®]

Root canal disinfection gel

Prescription

Desgel used for disinfection of the root canal as a bactericidal and hemostatic agent after instrumental processing of the root.

Composition and main characteristic

Desgel made on the base of disinfectant active agent, active to gram-positive and gram-negative bacteria. The preparation is stable and retains activity in the presence of blood and pus.

It has sufficient viscosity to process dental cavities of maxillary and have sparing effect on the oral mucosa.

Method of application:

A small quantity of gel put from the needle tip of the syringe directly into the root canal. Allow 2-3 minutes of exposure at the end of the time delay gel is washed off with water. Then, the

canal has to be dried with compressed air.


Then proceed to the filling of the root canal by selected filling material.


Storage conditions and labeling

Storage conditions

Desgel is stored in the manufacturer's packaging in closed, cool place, protected from atmospheric precipitation and direct sunlight, at a distance of at least 1 meter from heaters. To increase the shelf life is recommended to store it in the refrigerator at a temperature +5°C.

Labeling

The  symbol on the package show expire date.

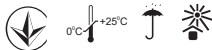

Latus PE, 61010 Textilna str.82, Khrkov, Ukraine.
Tel +380577332697. www.latus.com.ua