

Devilat[®]

Non-arsenic pulp devitalization doug

Prescription

Devilat paste intended for devitalization of the pulp in the case of treatment pulpitis by mortal extirpation or amputation, for the treatment of rudimentum root pulpitis in milk or molar teeth, as well as an additional tool for devitalization after repeated procedure after applying paste containing arsenic.

Composition and main characteristic

Included in the paste Devilat lidocaine hydrochloride and paraformaldehyde allow for prolonged devitalization of the pulp almost without irritation and pain and eliminate the for further processing of root to neutralize arsenic salts in case of use pastes which contain arsenic.

Method of application:

The necessary quantity of Devilat paste with a probe (without pressure) to enter into a pre-cleaned from carious dentin and an open cavity. The amount of paste is determined individually. In the treatment of pulpitis of milk teeth, as well as single rooted teeth in adults is enough to put the paste in an amount equal to the size of a millet grain. For pulp devitalization of multi rooted teeth amount of paste must be doubled. After applying the paste Devilat necessary to close hermetically the cavity by temporary filling material Tempolat. The complete devitalization of the pulp takes

place in 3-5 days. If the patient does not feel pain, permanent filling can be carried out within 24-48 hours after application of the paste. If there is no direct contact with the pulp devitalization should be carried out in two stages. Direct contact may be achieved only after the second stage reducing the viability of the pulp. After devitalization and extirpation of the pulp needed instrumental and pharmacological root treatment.

With increasing of pulpitis pain after the imposition Devilat paste (not opened pulp horn or paste imposed very tight) it is necessary to make infiltration anesthesia with lidocaine.


Storage conditions and labeling

Storage conditions

Devilat in packaged form should be stored in closed, cool place, protected from atmospheric precipitation and direct sunlight. To increase the shelf life prolonged storage it is recommended to use a refrigerator at + 5°C.

Labeling

The symbol on the package show expire date.

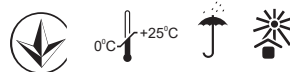

Latus PE, 61010 Textilna str.82, Khrkov, Ukraine.
Tel +380577332697. www.latus.com.ua