

# Iodotemp<sup>®</sup>-25

## Iodoformium paste

### Prescription

Paste **Iodotemp-25** is used in the case of acute and chronic periodontitis by temporary filling (2-3 days) root canal of the tooth for the purpose of relief on inflammatory processes and disinfection.

### Composition and main characteristic

Iodoform paste contains 25% of iodoformium, camphor, thymol; zinc oxide and paste-former. **Iodotemp-25** is plasticity paste with a characteristic smell of iodoformium. The paste is applied direct injection into the canal on the first or second visit, depending on the case, because **Iodotemp-25** excludes pre-medication treatment of canals. The paste does not harden because of this treatment can be recommence in the next time of your visit eliminating the risk of making the canal inaccessible as result of complete or partial filling.

### Method of application:

Uncover the pulp chamber, purified and expanded entry into the canals by applying the liquid to expand canals **Trimlat** or gel lubricant **Trimgel**. Washed with warm water and mild dried with compressed air. A small quantity of paste


**Iodotemp-25** introduced by canal-filler into the top part of the canal and then to the bottom by plugger, coat thickness should be about 0.2-0.5 mm. Remove excess of paste and filling with temporary cement (dentin paste). Next time visit remove the temporary seal, paste **Iodotemp-25** and, if necessary, to re-introduce **Iodotemp-25** and permanently filling up the canal.


### Storage conditions and labeling

#### Storage conditions

**Iodotemp-25** in packaged form should be stored in closed, cool place, protected from atmospheric precipitation and direct sunlight. To increase the shelf life prolonged storage it is recommended to use a refrigerator at + 5°C.

#### Labeling

The  symbol on the package show expire date.

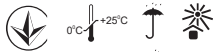

Latus PE, 61010 Textilna str.82, Khrkov, Ukraine.  
Tel +380577332697. [www.latus.com.ua](http://www.latus.com.ua)